


Packing the Smartview 360 CCTV

1. Your Smartview 360 will be packed into one box without the carry case and the carry case will be packed into a separate box. Place small piece of foam under the monitor to protect it. If you still have white film to cover the monitor, tape in place.

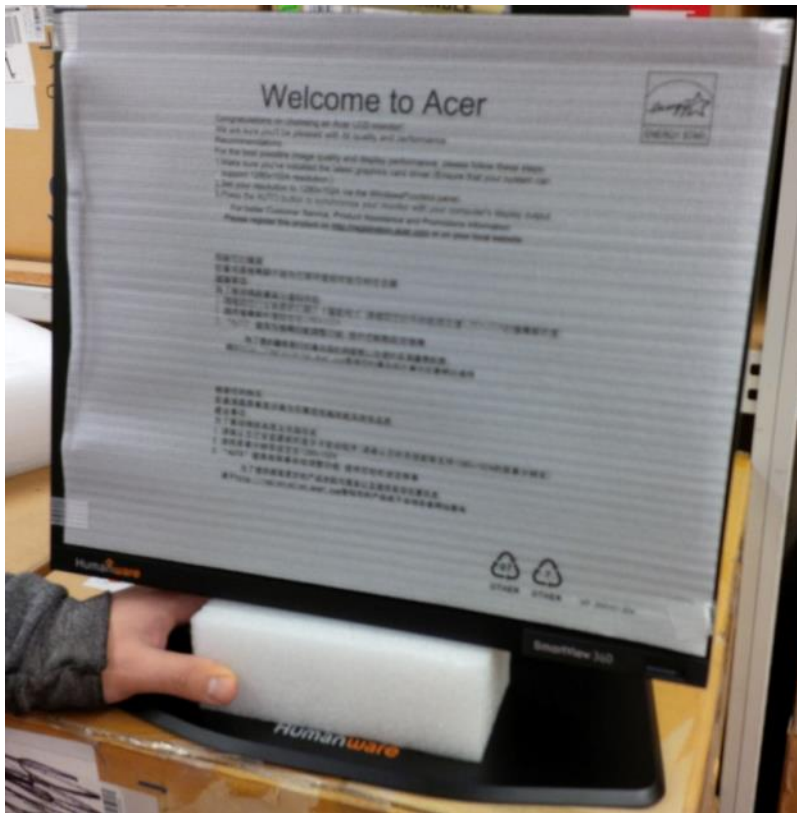

Sight Enhancement Equipment Pool & Assessment Centre
University of Waterloo, 200 University Ave. W., Waterloo, ON N2L 3G1
1-800-565-5965 or 519-888-4567 extension 37136

2. Make sure camera arm is parallel to monitor. Set monitor and camera into foam piece which has an indentation for camera and monitor.



3. Fit foam on other side of monitor and camera. Pack accessories into small box and fit accessories box into foam.



Sight Enhancement Equipment Pool & Assessment Centre
University of Waterloo, 200 University Ave. W., Waterloo, ON N2L 3G1
1-800-565-5965 or 519-888-4567 extension 37136

4. Cover CCTV in plastic bag.



5. Place CCTV into box. CCTV should be sitting upright with camera at top of the box.



Sight Enhancement Equipment Pool & Assessment Centre
University of Waterloo, 200 University Ave. W., Waterloo, ON N2L 3G1
1-800-565-5965 or 519-888-4567 extension 37136

6. Put cardboard piece over unit and tape box shut. Carry case should be packed in a separate box for shipping.

