

# Unpacking the Topaz Video Magnifier

---

1. The box is kept closed with four white connecting clips. The white clips can be found near the base of the box. There are 2 clips on 2 sides of the box.

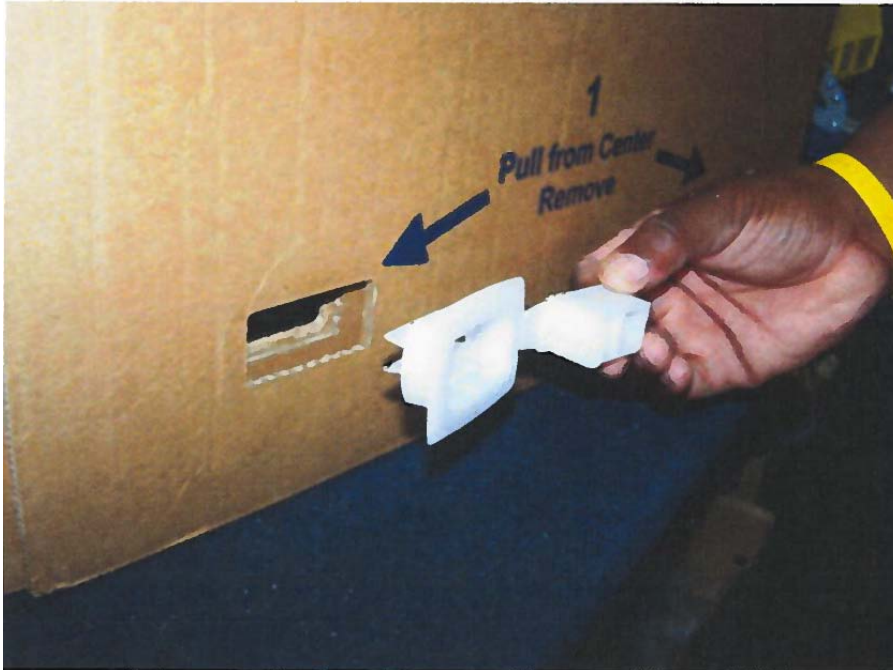


**Figure 1: White clips in a closed position.**

2. The clips snap open and then can be removed from the box completely, leaving a cut-out hole in the box. Hook your finger into the clip on the left hand side and pull until the clip opens. Then pull the entire clip from the box.



**Figure 2a: White clip after it has been snapped open.**



**Figure 2b: Clip being removed from the box.**

3. Once all 4 clips are fully removed from the box, lift the box straight up off the tray bottom.



**Figure 3: Box top being lifted off of box tray.**

4. There is a support made of foam and cardboard cradling the left and right side of the monitor. Remove both supports from the monitor. Remove both supports from the monitor.



**Figure 4a: Left and right support cradling the monitor**



**Figure 4b: Right support removed with only the left support remaining.**

5. The Topaz unit is enclosed in a plastic bag. Remove the plastic bag. The unit sits on a bottom tray of the box and the Topaz' base is cushioned by foam. The user documentation is sitting on the reading tray of the unit. Remove the documentation. Carefully lift the Topaz off of the tray. Lift the Topaz by the rear post (indicated by red arrow in picture), not by the monitor or reading lights. Place the Topaz where you will be using it.



**Figure 5: Topaz unit wrapped in plastic bag with documentation on the reading tray.**

Plug the power cord from the rear of the Topaz to an electrical wall outlet.

Please see your user manual for further instructions on how to use your Topaz unit. The documentation may include a warranty registration card. You do not have to return this card. Clients receiving a Topaz unit through the Assistive Devices Program automatically have a 5 year warranty against manufacturer's defects. If you experience problems with your Topaz, please call SEEPAC at 1-800-565-5965.