

Unpacking the Smartview 360 CCTV

1. Your Smartview 360 will come in 2 boxes. One box will have the case, and one will have the CCTV.



2. Lift wrapped CCTV out of the box.



Sight Enhancement Equipment Pool & Assessment Centre
University of Waterloo, 200 University Ave. W., Waterloo, ON N2L 3G1
1-800-565-5965 or 519-888-4567 extension 37136

3. Set CCTV on a flat, solid surface. Remove protective plastic wrapping.



4. Remove accessories box from the foam.



Sight Enhancement Equipment Pool & Assessment Centre
University of Waterloo, 200 University Ave. W., Waterloo, ON N2L 3G1
1-800-565-5965 or 519-888-4567 extension 37136

5. Remove foam from around the CCTV.



6. Remove remaining foam and film protecting monitor. Refer to the user manual for getting started with your new Smartview 360.

