

Unpacking the Clearview C CCTV

1. When you open your Clearview C box you will see the top is covered in foam. Remove the top pieces of foam.



2. You will now see the CCTV. At the top of the CCTV, beside the silver arm, there is a black handle folded into the unit.



Sight Enhancement Equipment Pool & Assessment Centre
University of Waterloo, 200 University Ave. W., Waterloo, ON N2L 3G1
1-800-565-5965 or 519-888-4567 extension 37136

3. Lift the CCTV out of the box using the black handle.



4. Set the CCTV on a flat, stable surface. There are 2 large pieces of foam supporting the monitor and camera.



Sight Enhancement Equipment Pool & Assessment Centre
University of Waterloo, 200 University Ave. W., Waterloo, ON N2L 3G1
1-800-565-5965 or 519-888-4567 extension 37136

5. Remove foam from around the CCTV.



Sight Enhancement Equipment Pool & Assessment Centre
University of Waterloo, 200 University Ave. W., Waterloo, ON N2L 3G1
1-800-565-5965 or 519-888-4567 extension 37136

6. Insert power supply cord into cord coming from support arm. Plug into the wall.



7. Remember to keep your box and packaging somewhere safe in case you need to return the CCTV for repairs. A fee of \$100 will be charged if the box is discarded.